ORALVedo

Termometro digitale orale.

GARANZIA ILLIMITATA

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0470

Numero verde per l'Italia 800 188 898

Digital electronic thermometer for oral use

Dear customer, thank you for buying **ORALVedo**, the digital thermometer for oral use which is innovative, quick and accurate.

The thermometer's characteristic flexible probe, with its unique hook-like shape, is easily and comfortably placed under the tongue to rapidly measure body temperature from 32.0 °C to 43.0 °C (90 °F - 108 °F).

All PiC Indolor thermometers are produced using the most advanced technology and undergo stringent quality control checks, in accordance with EU legislation, to guarantee the utmost quality and precision.

Note. Carefully read the instructions before use and keep for future reference.

Recommendations

Refrain from drinking hot or cold beverages, performing physical exercise, taking a bath or shower or smoking in the 15 minutes before

temperature measurement.

Before the measurement, relax and keep the mouth closed for five minutes.

Put a new cover on the probe before a measurement, to prevent the spread of germs and bacteria.

Do not force the probe, as excess pressure could damage it, making the warranty void.

Body temperature

Body temperature is very subjective and may change continually over the course of the day by as much as 1 °C. It is advisable to know your body temperature in normal healthy conditions, so you can tell if it changes.

Different types of thermometer (rectal, axillary, oral, frontal, auricular) will give different readings and it is hard to make correlations which are significant and objectively valid for each person.

It is therefore entirely normal that body temperature taken orally will be different from an axillary measurement and it is wrong to compare the two.

The table below shows typical temperatures for adults, based on different measurement sites:

MEASUREMENT SITE	"NORMAL" TEMPERATURE RANGE
Rectal measurement	36.6–38°C
Axillary measurement	34.7–37.3°C
Oral measurement	35.5–37.5 °C
Frontal measurement	35.8–38°C

Switching from C° to F° and vice versa

The thermometer can measure temperature in Celsius and Fahrenheit. To switch from one mode to another press the on/off button for 5 seconds when the thermometer is off, then press again to switch to the other scale.

Instructions for use

1. Put a new cover on the probe.
2. Press the on/off button. The last recorded temperature will appear on the display. No temperature will be displayed if it is the first time a measurement is being taken.
3. When "Lo" appears on the display, the thermometer is ready for use.
4. Place the sensor on the tip of the probe under the tongue, as close as possible to the tongue's hot spots (see figure 1). Keep mouth closed for a more effective measurement.
5. The measurement is finished only after 4 beeps are emitted (which last for approximately 10 – 15 seconds).
6. Remove the thermometer from the mouth and read the measurement on the display.


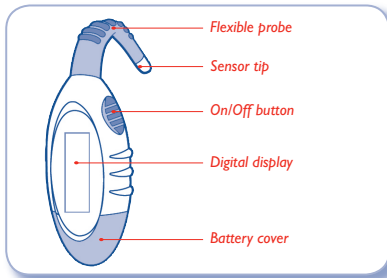
Figure 1

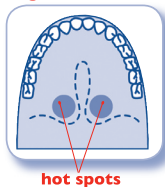7. To turn off the thermometer press the on/off button; if this button is not pressed the thermometer will automatically switch itself off after around 10 minutes.
8. Remove and throw away the cover.

Backlit display

The digital display lights up when the on/off button is pressed and when taking a measurement for easy temperature reading, even when ambient lighting is poor.

Probe cover

After each measurement the probe cover should be changed. This is important for hygiene, as it prevents the spread of germs and bacteria.

the probe covers, besides being supplied with the thermometer, are also available separately.

Important

Keep the thermometer out of reach of children.

Keep the battery out of reach of children, the elderly and pets (the battery is toxic if ingested).

Never leave children unattended when taking their temperature.

Make sure that the cover has not been swallowed during and after the measurement.

The thermometer must be stored at room temperature, in a dry place, away from heat and direct sunlight.

Strong electromagnetic fields could affect the proper functioning of the thermometer.

Non-waterproof.

Remove the batteries from the thermometer if not used for a long period of time.

If high temperatures persist, seek medical advice.

Cleaning and taking care of the thermometer

It's possible to clean the probe and the tip containing the sensor after use, with soap and warm water, or with a 70% ethyl alcohol solution.

Dry with a cloth.

Do not boil the thermometer and do not place in a dishwasher. This could damage the product and would make the warranty void.


Replacing the battery

If the battery is flat, a symbol will appear on the display . Replace the battery as follows:

1. Use a Phillips screwdriver to remove the

battery compartment cover on the back of the thermometer.

2. Remove the battery and dispose of in recycling containers.
3. Fit a new SR45 1.5V or equivalent battery, with the positive end (+) facing upwards and the negative end (-) facing downwards.
4. Fit the battery compartment cover back on and tighten the screws with the screwdriver.

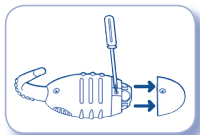Technical data

Measurement range: 32°C - 43°C (90°F - 108°F)

Laboratory accuracy: in compliance with EN12470:3:2000 ASTM E 112-00

Display: 4-digit, backlit display (dimensions: 22mm x 10mm)

Measurement time: 10-20 seconds

Storage: last temperature recorded

Temperature range: 10.0°C - 45.0°C

Number of possible measurements: more than 1,500

Powered by: SR45 1.5V battery (or equivalent)
BF-type applied part

Unlimited warranty

Artsana warranties **ORALVedo** against defects caused by faulty materials or workmanship. The warranty applies if the device is employed for normal domestic use in compliance with the indications in this leaflet.

Artsana may not in any case be held liable for particular, inherent or consequential damages, in relation to the use of this product or for costs greater than the original cost of the product.

This warranty does not cover any damages caused by the battery and accidental damages or damages resulting from improper use. Further warranties may vary from country to country.

(IT) This product conforms to the EU Directive 2002/96/EC.

The symbol on the thermometer (a  refuse container with a cross), indicates that the product at the end of its useful life must be handled separately from household waste and disposed of at a recycling centre for electrical and electronic equipment, or returned to the dealer when an equivalent product is purchased. The customer is responsible for disposing of this product as indicated above. Proper disposal, for subsequent recycling, pro-

cessing and environmentally-friendly disposal helps to avoid possible negative effects on the environment and on health and also helps to recycle the materials this product is made from. For more detailed information about waste collection systems, please contact your local waste disposal centre or outlet where you purchased this thermometer.

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