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#### GB What is it?

Saline solution is a sterile preparation of sodium chloride and purified water. The vial is sterile, practical and ready to use, and should be kept ready to hand. As there are no contraindications, it is suitable even for long-term use. The product is supplied in single-dose vials. To maintain sterility, it is recommended that the contents be used all at once.

#### INDICATIONS

Saline solution is indicated:  
• alone in aerosol therapy, to liquefy secretions to humidify nasal mucous membranes, for nasal cleanliness and

excess nasal secretions in adults and for children unable to blow their nose autonomously • for daily hygiene of nasal mucous membranes, to prevent congestion caused by build-up of secretions • to ease drainage of excess secretions caused by colds • can also be used for external cleaning of eyes.

#### INSTRUCTIONS FOR USE IN NASAL CLEANING

Open the sterile bottle by twisting the butterfly cap (fig. 1); Lean your head towards the left; for children, lay the child on one side, supporting the head with a hand (fig. 2). Insert the bottle spout gently into

the upper nostril and irrigate with plenty of solution. Repeat for the other nostril. Wait a few minutes for the solution to work. Remove secretions with a tissue; in the case of children, lift the child up and remove secretions with a tissue or nasal aspirator (fig. 3). Use as often as necessary or as instructed by your doctor.

#### INSTRUCTIONS FOR USE IN AEROSOL THERAPY

Open the sterile bottle by twisting the butterfly cap (fig. 1); Open the aerosol chamber by rotating anticlockwise (fig. 4); Add the required volume (fig. 5);

Close the aerosol chamber by rotating clockwise and attach the other necessary accessories (connector tube, mask and mouthpiece). Start the application.

#### INGREDIENTS

Sodium chloride 0.9%, sterile water to 100 ml.

#### SALES UNIT

REF 00 039101 020 100 -  
10 x 2.0 ml bottles  
REF 00 039101 050 100 -  
20 x 5.0 ml bottles  
REF 00 039101 100 100 -  
10 x 10.0 ml bottles

#### WARNINGS

For external use, do not inject. Store in a cool, dry place away from heat and light. Sterility guaranteed only if pack is intact and undamaged. Keep instructions until product is finished.

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