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GB VEDOBEEP digital electronic thermometer

Dear customer, thank you for buying **VEDOBEEP**, the innovative, quick and accurate digital thermometer.

All **PiC Indolor** thermometers are produced using the most advanced technology and undergo stringent quality control checks, in accordance with EU legislation, to guarantee the utmost quality and precision. The **VEDOBEEP** digital thermometer is highly shock-proof and quickly measures body temperatures in the range 32,0°C to 43,9°C (accuracy ±0,1°C).

Note: Carefully read the instructions before use and keep for future reference.

BODY TEMPERATURE

The thermometer must be positioned correctly to ensure an accurate reading.

Axillary measurement (requires longer measurement time): Ensure that the armpit is dry and that clothing does not impede the thermometer's correct positioning in the armpit. Place the end of the thermometer in the armpit and press the arm against the chest, in order to keep the thermometer in place and block any air currents.

Oral measurement: Place the end of the thermometer under the root of the tongue, ensuring that the metal sensor is in good contact. Keep the mouth closed while taking the reading and breathe through the nose only. Do not eat or drink anything for 15 minutes before the reading. **Rectal measurement (especially indicated for babies and children):** gently insert the thermometer into the anus to a distance of no more than 0,5 cm. If there is any kind of resistance, stop immediately. Keep the thermometer in place during the reading.

INSTRUCTIONS FOR USE

To turn on the thermometer press the on/off button; a short beep will indicate that the thermometer is on. The display automatically runs a test to check that it is functioning properly, briefly displaying all symbols. Once you have removed your finger from the button the display will read 'ok' and then 'Lo' with a flashing '°C' symbol; the thermometer is now ready to take a measurement and so it should immediately be positioned as desired. If the thermometer is not working correctly the display will show 'Err'. Measurement time varies according to the area of the body where it is being taken (around 2 minutes under the armpit, approximately 1 minute for oral and rectal measurements). An alarm will sound if the measured temperature is more than 37,5°C; a different sound indicates that the reading has been taken. To turn off the thermometer press the on/off button; if this button is not pressed at the end of measurement the thermometer will automatically switch itself off after around 10 minutes.

READING THE MEMORY

With the thermometer turned off, press the on/off button; keep the button pressed down to display the last temperature reading taken.

CHANGING FROM °C TO °F

The thermometer can take temperatures in both Celsius and Fahrenheit. To change from one to the other, with the thermometer turned off press the on/off switch twice and then press for at least 6 seconds. Repeat to change back to the original unit.

IMPORTANT

Keep the thermometer out of reach of children. Keep the battery out of reach of children, the elderly and pets (the battery is toxic if ingested). Never leave children unattended when taking their temperature. The thermometer must be stored at room temperature, in a dry place, away from heat and direct sunlight. Strong electromagnetic fields may affect the proper functioning of the thermometer. Remove the batteries from the thermometer if not used for a long period of time. If high temperatures persist, seek medical advice.

CLEANING AND MAINTENANCE

The thermometer must be cleaned with a wad of cotton soaked in alcohol or disinfectant. Do not boil or place in the dishwasher, as this could damage the product and would make the warranty void.

REPLACING THE BATTERY

When the battery is flat the symbol  will appear on the side of the display and no more readings can be taken. Replace the battery as follows:

- Open the battery compartment by pressing lightly on the cover on the side opposite the display.
- Remove the battery with a suitable non-metal utensil.
- Insert the new LR41 or equivalent battery with the positive end (+) towards the display.
- Close the battery compartment.
- Dispose of the old battery in a battery recycling bin. Warning: toxic if swallowed.

TECHNICAL DATA

Type:	Electronic digital thermometer for measurement of body temperature
Measurement range:	From 32.0 to 43.9°C Temp. Less than 32.0°C: "Lo" Temp. Above 43.9°C: "Hi"
Laboratory accuracy:	± 0.1°C in conformity with EN 12470-3:2000 ASTM E 112-00
Display:	Liquid crystal (LCD) with 1/10 resolution
Acoustic signal:	End of measurement and on pressing the on/off button
Fever alarm:	On measuring temperature >37.5°C
Memory:	Last reading taken
Conditions of use:	Room temperatures between 10°C and 45°C. Humidity 30% to 85%
Storage conditions:	Room temperatures between -25°C and 55°C. Humidity 15% to 95%
Battery:	LR41 1.5V
Number of possible measurements:	More than 4.000
Waterproof:	In compliance with EN12470-3:2000

ELECTROMAGNETIC COMPATIBILITY

The device is manufactured according to applicable standards and complies with all requirements on protection relating to electromagnetic compatibility; special precautions should however be taken for EMC and the device should be installed and operated in compliance with the information on EMC in this leaflet. Portable and mobile radio communication equipment may affect the operation of the electromagnetic device. When electromagnetic disturbance occurs, repeat temperature reading if values are uncertain.

Guide to Manufacturer's Declaration – Electromagnetic Emissions		
The VedoBeeep digital electronic thermometer is designed to operate in the electromagnetic environment indicated below. The customer or user of the thermometer should ensure the device is used in this environment.		
Emissions test	Conformity	Electromagnetic environment - guide
RF emissions	Group I	The VedoBeeep thermometer uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions	Class B	The VedoBeeep thermometer is suitable for use in all premises, including home use and rooms connected directly to a low voltage mains supply for domestic use.

UNLIMITED WARRANTY

Artisana guarantees this product against any defects due to faulty materials or manufacture.

The warranty applies if the device is employed for normal domestic use in compliance with the indications in this leaflet.

Artisana may not in any case be held liable for specific, inherent or consequential damages in relation to the use of this product or for costs greater than the original cost of the product.

This warranty does not cover any damage caused by the battery and accidental damage or damage resulting from improper use.

Further warranties may vary from country to country.

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