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GB VEDO LUX electronic digital thermometer

Dear customer,

Thank you for buying the **VEDO LUX** digital thermometer.

All **PiC Indolor** thermometers are produced using the most advanced technology and undergo stringent quality control checks, in accordance with EU legislation, to guarantee the ut- most quality and precision.

The **VEDO LUX** digital thermometer is able to rapidly measure body temperature from 32°C to 43.9°C (tolerance +/- 0.1°C). The special flexible probe ensures maximum comfort during measurement. It is produced with materials that make it both shock and water-resistant.

INSTRUCTIONS FOR USE

To turn on the thermometer press the on/off button; a short beep will indicate that the thermometer is on.The display illumi- nates and automatically runs a test to check that it is functioning properly, briefly displaying all symbols.

Once you have removed your finger from the button the dis- play will read 'ok' and then "Lo" with a flashing "°C" symbol; the thermometer is now ready to take a measurement and so it should immediately be positioned as desired.

If the thermometer is not working correctly the display will show "----".

Measurement time varies according to the area of the body where it is being taken (around 2 minutes under the armpit, ap- proximately 1 minute for oral and rectal measurements).

A beep will indicate that temperature has been taken.

To turn off the thermometer press the on/off button; if this but- ton is not pressed at the end of measurement the thermom- eter will automatically switch itself off after around 10 minutes.

BACKLIT DISPLAY

The display lights up automatically every time the on/off but- ton is pressed.

Keeping the button pressed down, once measurement has been made, the display lights up before turning itself off making it easy to read the temperature even in poorly lit areas.

TO ENSURE ACCURATE MEASUREMENTS

The correct positioning of the thermometer is essential for ob- taining an accurate reading.

Oral measurement: position the tip of the thermometer under the tongue towards the base, making sure that the metal sensor is in complete contact with the tissue. Keep the mouth closed during the measurement, breathing only through the nose.

Never eat or drink anything in the 15 minutes before you take the reading.

Axillary measurement (Requires more time): make certain that the armpit is dry and that clothes do not prevent the correct positioning of the probe inside the armpit.

Position the tip of the thermometer inside the armpit and press the arm against the chest to keep the thermometer in position and prevent any airflow.

Rectal measurement (Particularly indicated for children and infants): delicately insert the thermometer into the rectum to a depth of not more than 0.5 cm. If you perceive any type of resistance, stop immediately.

Hold the thermometer in position while taking the measure- ment.

READING THE MEMORY

With the thermometer off, hold down the ON/OFF button. After approximately two seconds, the last reading is displayed along with the symbols "M" and "°C".

REPLACING THE BATTERY

The thermometer is powered by a SR41 battery (Energi- zer 392, Duracell D392, Varta V392, Maxell SR41W, etc.). This type of battery will enable you to take more than 2000 measure- ments.

The thermomter's battery compartment has a special safety opening.

If the battery is flat, the symbol ■ will appear on the display. To replace the battery: slide out the lid of the battery compart- ment by pressing slightly towards the side opposite the display. Remove the battery using a pointed, non-metallic tool. Insert the new battery with the + side facing toward the display.Finally, close the battery compartment.

Flat batteries must be disposed of in specially differentiated waste collection containers.They are toxic if swallowed.

CLEANING AND DISINFECTION

The thermometer can be cleaned using a ball of cotton wool soaked in alcohol or a disinfectant solution.

TECHNICAL SPECIFICATIONS

Description:	Thermometer to measure body temperature
Measurement range:	From 32.0 to 43.9 °C Temp. lower than 32.0 °C: "Lo" Temp. higher than 43.9 °C: "Hi"
Precision:	+/- 0,1°C
Display:	Liquid crystal display (LCD). Mini- mum unit of measure displayed 0.1 °C
Acoustic signal:	Beeps when measurement is com- plete and whenever the ON/OFF button is pressed
Memory:	Displays the last measurement read- ing
Operating conditions:	Temperature from 10°C to 45°C Humidity from 30% to 85%
Storage conditions:	Temperature from -25°C to 55°C Humidity from 15% to 95%
Battery:	Type: SR 41 (Energizer 392, Duracell D392, Varta V392, Maxell SR41W, etc.)
No. measurements possible:	More than 2,000 measurements
Waterproof:	Yes
Standards:	EN 12470-3:2000 ASTM E 1112-00

WARNING: KEEP THE INSTRUCTIONS AND RE- FERTO THEM BEFORE USE

The thermometer must be stored at room temperature, in a dry place, away from sources of heat and out of direct sunlight. Strong electromagnetic fields could prevent correct function- ing.

Keep the thermometer out of the reach of children.

Never leave children alone while taking a temperature measure- ment.

If the temperature rises, consult your doctor.

This unit complies with EEC Directive 93/42.

UNLIMITED WARRANTY

Pic guarantees Vedo Lux for life from the date of purchase.The warranty does not cover damage caused by improper use. Any claim for damages covered by the warranty must be accompa- nied by the purchase receipt (cash register receipt).

This product complies with EU Directive 2002/96/EC.

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar ap- pliance. The user is responsible for taking the appliance to a special waste disposal site at the end of its life. If the disused appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both the environment and health, and contributes towards the recycling of the product's materials. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

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