DIAGNOSTIC

CSI 610

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Features:


- Smart Inflate technology
- Customisable dual memory
- 50 + 50 memories
- Average of last three readings
- Date and time
- One Touch easy measurement

Introduction

Blood pressure measurements taken with the CS 610 are equivalent to those obtained by a trained person using the armband/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers. The CS 610 is designed for home use by adults.


**Caution:**

Please read this manual carefully before use. For specific information on your own blood pressure, please contact your doctor. Keep this manual in a safe place for future reference.

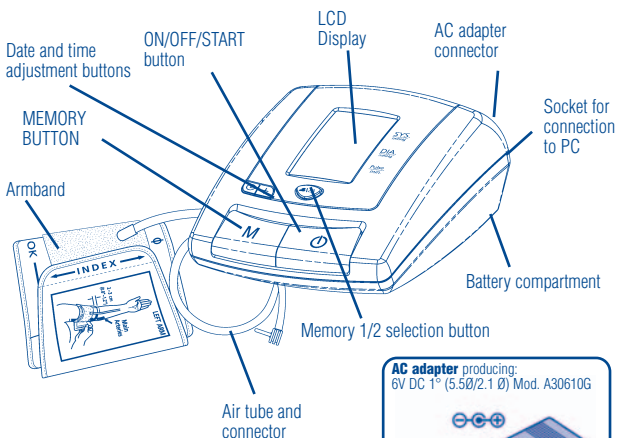Name/Function of each part:

4 x "AA" size 1.5 V batteries

AC adapter producing:
6V DC 1° (5.5Ø/2.1 Ø) Mod. A30610G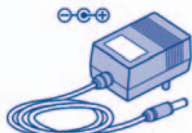

Smart Inflate measuring technology

This appliance uses the Smart Inflate technology to measure blood pressure. At the press of a button, the armband is automatically inflated. As it inflates, the CS 610 will measure your arterial blood pressure, analysing its oscillations. Once the measurement has been taken, the unit will deflate the armband and display the systolic and diastolic pressure and pulse rate values. The time required for measurement is much shorter, with consequent increased precision of the reading and comfort of the user.

Preliminary remarks

This sphygmomanometer complies with European regulations and bears the EC mark "CE 0068". The quality of the unit has been verified and conforms with the provisions of European Council directive 93/42/EEC dated 14th June 1993 regarding medical equipment, as well as EMC directive 89/336/EEC:

EN 1060-1 (Nov. 1995)	Non-invasive blood pressure measuring equipment. General requirements
EN 1060-3 (Feb. 1997)	Non-invasive blood pressure measuring equipment. Supplementary requirements for electromechanical blood pressure measuring systems

To ensure accurate measurements, the calibration of the unit should be checked every two years.

Arterial blood pressure

What is arterial blood pressure?

Arterial blood pressure is the pressure exerted on the artery as blood flows through it. The pressure measured when the heart contracts and pumps out blood is the systolic pressure (highest). The pressure measured when the heart dilates and blood flows back is called diastolic (lowest) blood pressure.


Why Measure Your Blood Pressure?

Amongst the various health problems afflicting modern man, problems linked to high blood pressure are by far the most common. The dangerously close relationship between high blood pressure and heart disease, characterised by high morbidity, has made it necessary to monitor blood pressure in order to identify those at risk.

Blood pressure standard

The World Health Organization (WHO) has established a blood pressure standard, identifying risk areas for hypotension (low blood pressure) and hypertension (high blood pressure). This standard, however, is a general guideline as individual blood pressure measurements vary according to population, age group, etc.


It is important to consult your doctor regularly. Your doctor will be able to tell you your normal blood pressure range as well as the point at which you should consider yourself at risk.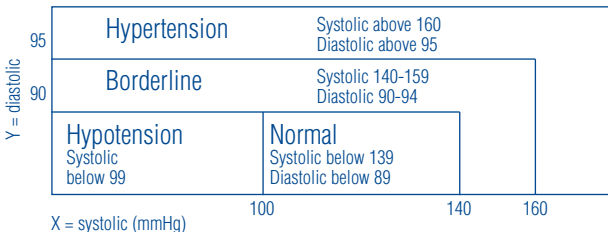


Blood Pressure Fluctuation

Blood pressure fluctuates all the time!

Do not worry if two or three measurements are high. Blood pressure fluctuates during the month and even throughout the day. It is also influenced by season and temperature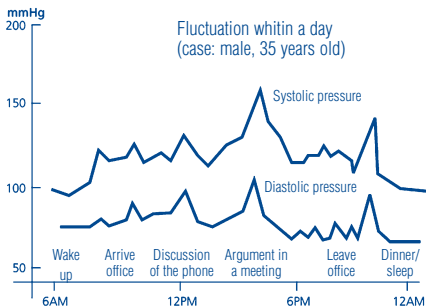

Key to display Display:

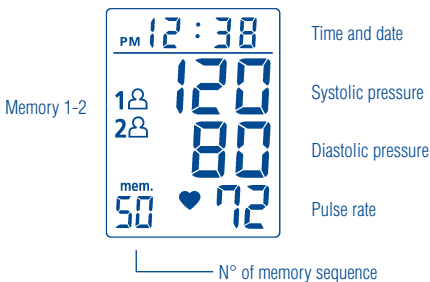Symbols

Showing when the user 1 memories are displayed

Showing when the user 2 memories are displayed

Showing when batteries need replacing

Showing the pulse rate per minute


mem.

Showing the number of readings memorised

Showing if a mistake is made during measurement

Dual power supply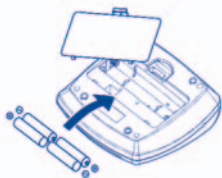


Inserting batteries

1. Insert or replace 4 "AA" size batteries in the battery compartment, according to the diagram shown on the inside.
2. Replace the cover of the battery compartment by placing the end tabs of the battery cover in the slots on the top of the battery compartment and pushing down the other end to secure the battery cover.

Replace the batteries when:

1. The flat battery symbol appears on the display.
2. Nothing appears on the display when the ON/OFF/START button is pressed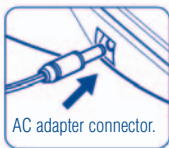

AC adapter


Mod. A30610G

AC adapter connector. This appliance can also be powered by an AC adapter. The AC adapter should be connected as shown in the diagram (left).


Note: Batteries are hazardous waste. Do not dispose of them together with domestic refuse.

Applying the armband


1. Connect the armband to the unit.
2. Open the armband, leaving the "Index" end in ring D.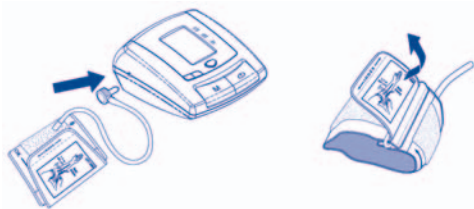

3. Apply armband to left arm. The "OK" interval mark should be turned towards you, whilst the rubber tube should be positioned in the same direction as your arm. Place the artery mark Φ on the main arteries (on the inside of the arm) on the upper part of the arm. Tighten the armband by pulling the "Index" end.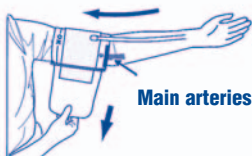

4. Turn the palm of your left hand upwards and position the edge of the armband about 2-3 cm above the inside of your elbow.

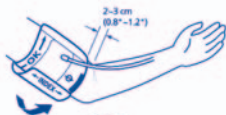5. **If the Index line falls within the OK interval, shown on the upper edge of the armband, then this armband is suitable for you.** If, on the other hand, the "index" line is situated outside the OK interval, you may require a special armband.

Taking a measurement


1. Press the memory selection button to choose the archive in which you wish to store the reading.

2. Press the ON/OFF/START button. The entire display will be shown for about two seconds before returning to "0".

3. The unit will automatically start inflating the armband to measure your blood pressure. During inflation the display will show a variable reading of the pressure inside the armband. **It is very important to remain still and quiet during measurement. The slightest movement could result in an inaccurate measurement.**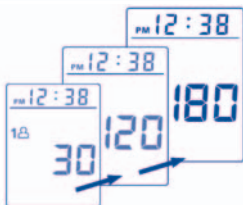

4. When the measurement is complete, the systolic and diastolic pressure and pulse rate will be shown simultaneously and automatically saved in the chosen archive. Each of the two memory archives can store up to 50 measurements.
5. Press the ON/OFF/START button to switch the unit off. If no button is pressed, the unit will turn itself off automatically after 1 minute.
6. To interrupt the measurement, press any button. The armband will deflate automatically.


1. The appliance has two memory archives. Each memory archive can contain up to 50 measurements.
2. Press the memory selection button to display the desired data archive. Press the MEMORY button to display the values with the relative sequence numbers.
3. The first value displayed is an average of the last three measurements taken. Each subsequent press of the MEMORY button will display the measurements stored in the memory, starting with the most recent one.

Deleting values from the memory

1. Press the memory selection button to access one of the two archives.
2. Press the MEMORY button for about 5 seconds to delete all the information stored in the archive.


Press the MEMORY button for 5 seconds to eliminate all the data in the memory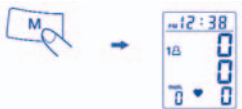

Please note: The data contained in both archives may be lost if any of the batteries are removed. An optional program is available that enables the data memorised by the appliance to be uploaded onto a PC.

1A ↔ 2A

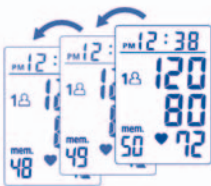Press the "C" button

Month flashes

Increase value

Confirm value

Adjusting the clock

1. To adjust the date and time displayed by the appliance, press the "C" button. A flashing number indicating the month will appear on the display.
2. Adjust the month by pressing the "+" button. Each time the button is pressed the value will increase by one unit. Press the "C" button to confirm the selected value. The screen will show a flashing number indicating the day.
3. Adjust the date, hours and minutes as described in step 2, using the 2 "+" button to adjust the values and the "C" button to confirm them.
4. The number "0" will appear when the sphygmomanometer is ready to start measuring.

Helpful tips

Here are a few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after eating a large meal.
- To obtain more accurate readings, wait at least one hour before measuring your blood pressure.
- Do not smoke or consume alcohol before measuring your blood pressure.
- Do not measure your blood pressure if you are physically tired.
- It is important to relax during measurement. Try to rest for 15 minutes before measuring your blood pressure.
- Do not measure your blood pressure if you are tense or under stress.
- Measure your blood pressure at normal body temperature. If you are feeling hot or cold, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), put it in a warm place for at least an hour before use.
- Leave at least 5 minutes between measurements

Troubleshooting

If any faults should occur during use, please check the following points:

Symptom	Check point	Check point
Nothing appears on the display when the ON/OFF/START button is pressed	Have the batteries run down?	Replace them with 4 new alkaline batteries.
	Are the batteries inserted correctly?	Reinsert the batteries in the correct position.
The EE symbol appears on the display or the blood pressure value is excessively high or low	Is the armband correctly applied in the proper position?	Reapply the armband in the correct position
	Did you talk or move during measurement?	Take the measurement again without moving.
	Did you move your wrist whilst wearing the armband?	

Note: If the unit still does not work, take it to an authorised Artsana service centre. Under no circumstances should you disassemble and attempt to repair the unit yourself.

Warnings:

1. The appliance contains high-precision components. Therefore avoid extreme temperatures, humidity and direct sunlight. Avoid dropping or vigorously shaking the main unit and protect it from dust.
2. Clean the monitor body and armband carefully with a soft and slightly damp cloth. Do not press. Do not wash the armband or use chemical cleaners on it. Never use thinner, alcohol or petrol as cleaning fluids.
3. Leaky batteries can damage the unit. Remove the batteries if the unit is not used for long periods.
4. For safety reasons do not let children use the appliance.
5. If the unit is stored at temperatures near freezing point, allow it to acclimatise at room temperature before use.
6. The CSI 610 is not field serviceable. Do not attempt to open the appliance using tools or try to adjust anything inside it. If you have any problems, please contact an authorised Artsana service centre.

7. The unit is not suitable for patients with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, circulatory disorders or kidney problems, those who have suffered a stroke or are in a state of unconsciousness.
8. Do not use the appliance in the vicinity of devices that use radio frequencies, such as mobile telephones.

Specifications*:

Measurement method:	oscillometric
Measurement range:	pressure: 20-280 mmHg; pulse 40-180 beats per minute
Pressure sensor:	Semiconductor
Accuracy:	blood pressure: ± 3 mmHg; pulse: ± 5 % of reading
Inflation:	pump driven
Deflation:	automatic pressure release valve
Memory capacity:	50 x 2
Auto shut-off:	1 minute after last button was pressed
Operating environment:	temperature: 5 - 45 °C (41 - 113 °F); relative humidity: 40 - 85 %
Storage environment:	temperature: -10 °C – 60 °C (14°F - 140°F); relative humidity: 10 – 95%
DC power source:	4 x LR6 (AA) batteries 1.5V DC 6V 1A DC
AC power source:	230V AC 50 Hz 0.2 A (plug dimensions: internal (-) \varnothing 5.5 external (+) \varnothing 2.1) MOD – A30610G
Dimensions:	137(L) x 140(W) x 63(H) mm
Weight:	450 g (gross weight) (without batteries)
Arm circumference:	adults: 24 – 36 cm (9.4" – 14.2") (standard) large: 34 – 48 cm (13.4" – 18.9") (optional)
Max. usage time:	max. 2 min. BF type classification

* Specifications are subject to change without prior notice

Blood pressure log:

Name:	Age:					Weight:				
Date										
Time										
mmHg										
220										
200										
180										
160										
140										
120										
100										
80										
60										
Pulse										
Body Condition										

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