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**EN WHAT IT IS**  
RinoFlux isotonic saline solution is a medical device for nasal rinsing, aerosol therapy and rinsing the eyes externally. When used for nasal rinsing (including with a nasal irrigation device) it cleanses and moisturizes the mucosa helping drain secretions and eliminate excess mucus, in the event of infections of the upper airways. Helps prevent otitis and rhinosinusitis. When used in aerosol therapy it moisturizes the airways helping to expel sputum, in the event of infections of the lower airways. The external cleaning of the eyes helps prevent viral or bacterial infections.

**INSTRUCTIONS**  
**• IN AEROSOL THERAPY**  
Twist the tab to open the vial (fig.1); RinoFlux can be used with any aerosol device (fig.5). Comply with the instruction manual for the given device.

**• FOR NASAL IRRIGATION**  
Twist the tab to open the vial (fig.1); lean your head to the left or, for children, lay the child down on one side supporting the head with your hand (fig.2), gently rest the opening of the vial against the upper nostril, press lightly and irrigate the nostril abundantly. Wait a moment for the solution to act. Clear any secretions with a tissue. For children, lift the child and clear

any secretions with the nasal aspirator or a tissue (fig. 3). Turn the head and repeat the operation with the other nostril. For use with a nasal irrigation device, follow the instructions for the given device.

**• FOR EXTERNAL CLEANING OF THE EYES**  
Twist the tab to open the vial (fig.1); soak a sterile gauze pad (one per eye) and gently clean the eyelid externally avoiding direct contact with the eyes (fig. 4).

**COMPOSITION**  
Sodium chloride 0.9 g, sterile water as required for 100 ml.

**! WARNINGS**  
Solution not to be used for injection. For nasal use, do not ingest. Use as instructed by a doctor/ paediatrician. Store in a cool and dry place, do not expose to sources of heat and light. Do not use after the expiration date, as sterility is no longer guaranteed. Do not use the same single-dose vial to clean both the nose and eyes or with another patient. For children less than 2 months of age, or patients with significant respiratory illness, or in the event of frequent nose bleeds, or neurological or musculoskeletal diseases or maxillofacial trauma consult your physician before use. Report any serious accident occurring in

connection with the device to your competent authority and to the manufacturer. Keep out of reach of children. Do not use if the packaging is damaged.

Date of manufacture: first 4 digits of LOT (e.g. 23039999; 23=2023; 03=March)

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